

# ConComp®

## Container Compatibility

Fluid Administration

**ConComp®** Container Compatibility  
Active ingredient - Container - IV solution

Code

Login Name

Password

**ConComp®** Container Compatibility  
Active ingredient - Container - IV solution

[www.concomp.com](http://www.concomp.com)

Your personal link to a powerful easy-to-use compatibility database

**B. BRAUN**  
HOSPITAL CARE  
B. Braun Melsungen AG  
D-34 209 Melsungen

ConComp®, a unique information source.  
For all your compatibility questions on drugs,  
container material and I.V. solutions.

# How to Guide ...

... through the powerful easy-to-use compability database.

## Fluid Administration



1

Ensure you have your ConComp® ID card with your pass code ready. If you have no ID card yet [Apply online](#) at [www.concomp-partner.com](http://www.concomp-partner.com).



2

Enter the following web address [www.concomp-partner.com](http://www.concomp-partner.com).



3


If you have entered the site before, please fill in your Login Name and Password. Please proceed to stage 7.



4

### Important for new users:

If this is the first time you have entered the site to Login, please go directly to the ConComp® Logo to the far right of the screen and click on the words [First login](#).

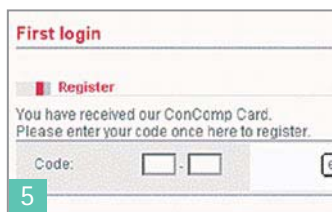

5

### First Login.


As requested, please enter your 8 letter pass code – taken directly from ConComp® ID card. Then please press [enter](#). For the next login, please use your new defined Login Name and Password.



6

### Edit your profile.

- Enter your personal data (name, hospital, position etc.)
- Enter your login name (this can be your first name)
- Enter your password (use something that you will remember easily)
- Re-enter your password for confirmation
- When all of the above are completed, please press [Save Userdata](#)

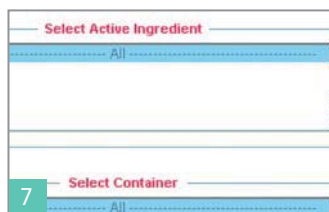

7

### Stability and Compatibility Data.

To check the stability and compatibility of your requested drug with a container and solution, simply highlight one of the alphabetically listed drugs in the 'Active Ingredient' box. Once your drug is selected, the choice of container and I.V. solution is required. Now highlight one from each of the 'Container' and 'Solution' boxes.



8

### Your articles.

Once your selection for all three components has been made, the relevant research articles will be listed below. Please click the box next to your selected research article(s), then press [Showarticles](#). The articles can be easily read on screen or printed out for future reference.



**B | BRAUN**  
SHARING EXPERTISE

B. Braun Melsungen AG

P.O. Box 1120  
D-34209 Melsungen  
Tel +49(0) 56 61 71-0

[www.bbraun.com](http://www.bbraun.com)  
[www.concomp-partner.com](http://www.concomp-partner.com)